




## ***Cochlear Mini Microphone 2+***

**Turn on the MM2+:** Press and hold the ON/OFF button located at the top of the microphone until the LED light turns *green* (*red* for off).

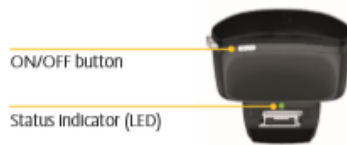

**Activate streaming:** Your student will initiate streaming to the microphone via their processor/s.



The MM2+ has a range of approximately 25 feet and is dependent on line of sight. When it is turned off or out of range for more than five minutes, it will automatically stop streaming. Streaming will then need to be reinitiated via the processor/s.

**Placement of the MM2+:** Once the MM2+ is turned on and streaming to the processor/s, place the microphone within 6-10 inches of the speaker's mouth. Please ensure that clothing and jewelry are not touching the microphone as it is very distracting to the listener.



The MM2+ can also be placed at the center of the table during group work. This will automatically activate the multi-directional microphones in the device.

**Muting the MM2+:** The MM2+ should be muted whenever the speaker is not addressing the student or class. The mute button is located at the side of the device. The indicator light will flash orange when it is muted. *It is recommended to mute the MM2+ rather than turning it off throughout the day as turning it off will cause it to disconnect from the processor/s.*



**Connecting to external audio:** The MM2+ can be plugged into personal audio (i.e. iPad, Chromebook) using the provided 3.5 mm audio cable. The MM2+ can be placed near a speaker for classroom audio.

**Charging the MM2+:** The MM2+ will need to be plugged in and charged at the end of each school day. It is fully charged when the LED turns solid red with green flashes every two seconds.

**Troubleshooting the MM2+:** For additional information on the MM2+, please refer to: <https://www.cochlear.com/us/en/home/ongoing-care-and-support/device-support/nucleus-family/nucleus-7>. Alternatively, feel free to contact your educational audiologist.

Melanie Monaghan, R. Aud, Aud (C)  
Educational Audiologist  
Eastern Edge Regional Collaborative Service Delivery (EE-RCSD)  
683 Wye Road, Sherwood Park, T8B 1N2  
Melanie.monaghan@eips.ca