

Inclusive Education

HOW-TO: LAMP Vocab Air Dropping

To keep a student's LAMP Words for Life system consistent and reliable, it's important that all devices stay updated with the same vocabulary file. The best practice is to make all changes on one device, ideally the **model talker**, and then air drop the updated file to the student's device(s). This prevents lost vocabulary additions/changes and avoids frustration caused by having different words available on different devices.

After you finish any editing—this includes copying a vocabulary or Air dropping it—please make sure to turn editing locks back on so the system remains secure and stable. **If a lot of changes and/ or additions have been made, rename this vocabulary using the school district's standard naming conventions (0 – the student's first name and then last name Initial – Month2025 (e.g. 0 - KrisM – Dec2025)**

Air Dropping a Vocabulary to Another iPad

1. Turn on receiving iPad and turn off its guided access, ensure Air Drop is on both iPads.
2. On the iPad Talker with the vocab to be transferred, select "Vocab" from the left side of the banner
3. Choose new vocab
4. Select "Menu" from the right side of the banner
5. Select "Import/Export Vocab"
6. Select "Export and Share"
7. Select the vocab; select Done
8. Choose Air Drop; select destination iPad talker
9. Accept the vocab on the receiving iPad
10. Exit out of Menu as needed
11. Turn Menu Passcodes and Guided Access back ON

To load the new vocabulary onto the other Talker, follow these steps:

1. Go to Vocab and press Choose New Vocab
2. Go to Menu (top right of the page) and select Import/Export Vocab
3. Select Import from Files
4. A File Menu will pop up; sometimes you will see the new file right away but sometimes you have to go to Recently Shared and find it.
5. Select the file and press Open
6. Then Select import